# Vitrification Cryotop Method for Oocyte and Embryo Thawing Media

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**Thawing Media**  
No. 1 (Color: Red) Thawing Solution (TS): 2x4.0 ml vials  
No. 2 (Color: Yellow) Diluent Solution (DS): 1x4.0 ml vial  
No. 3 (Color: White) Washing Solution (WS): 1x4.0 ml vial  
**Caution:** Before use, check the specifications of the container and labeling number marked on top of the cap, and color, label, labeling color, solution name, and volume. If you notice anything unusual regarding the items mentioned above, do not use the product and please contact the distributor.

**Recommended equipment**  
• Petri Dish: 35 mm for TS  
• ReproPlate with 6 wells  
**Caution:** This product is sterilized. Please handle the product under sterilization in a clean bench.

**Instructions for use (IFU)**  
**Preparation**  
No. 1 (COTTON) seal with a Petri Dish in an incubator to 37°C (1.5 hours). Pour the full content of TS into the Petri Dish.  
Bring DS and WS to room temperature (23-27°C as recommendation).  
Drop each 300 µl of DS into first well, WS into second and third well on the Repro Plate with a pipette.  
**Caution:** Use a sterilized pipette as a handling tool, with a suitable internal diameter for oocytes or embryos. The recommended internal diameters are as follows: 180 µm for oocytes (MI), 180 µm for pronuclear stage embryos, and 250 µm for cleavage or blastocyst stage embryos.

**Thawing**  
Step 1: Quickly immerse the Cryotop strip completely into the TS. Leave it for 1 minute.  
Step 2: Aspirate the oocyte (MI)/embryo with the pipette and gently place it on the BOTTOM of the DS. Leave it for 3 minutes.  
Step 3: Aspirate the oocyte (MI)/embryo with the pipette and gently place it on the TOP of the WS third well. Repeat this process one more time.  
Step 4: Aspirate the oocyte (MI)/embryo with the pipette, gently place it on the TOP of the WS third well. Repeat this process one more time.  
Step 5: Transfer the oocyte (MI)/embryo to a culture dish containing the appropriate culture media. Incubate the oocyte (MI)/embryo in a 37°C incubator to complete recovery.

**Caution:** It is recommended to incubate oocytes (MI) and embryos for 2 hours.  
**Caution:** Wash the embryo and incubate embryo for recovery in appropriate media to avoid the WS of third well to be transferred in patient body.

**Quality control specification**  
The following tests were performed for each lot of this product:  
Sterility by Sterility Test (EP)  
Endotoxin by LAL methodology  
Mouse Embryo Assay (One Cell)  
pH (EP)  
Osmolality (EP)

**Storage instructions and stability**  
Store the vials at 2 to 8°C  
This product is stable until the expiry date shown in the labeling on the vial.

**Composition**  
• HEPES within Basic Culture Medium  
• Tris  
• Hydroxypropyl Cellulose  
• Gentamicin

**Contraindication**  
• Do not re-sterilize.  
• Do not re-use. Reuse may cause contamination.  
• Do not use solution that shows cloudiness or becomes discolored.  
• Do not use the product if you notice anything unusual regarding the specifications on the label (number, color, name, volume).  
• Devices sterile if the package or container is unopened or undamaged.  
• Do not use if the package or container are opened or damaged.  
• Upon delivery media must be stored in original unopened container and refrigerated at 2-8°C.  
• Do not use and please discard if the media were not stored under refrigeration (2 to 8°C).  
• Do not use the product if past the expiration date.

**Warning**  
Read the instructions for use prior to use.  
This product is intended to be used by medical specialists trained in fertility treatment.  
Aseptic technique should be used.  
Use sterilized equipment and materials only.  
Observe all federal, state and local environmental regulations when discarding the product.  
In case of eye or skin contact with Vitrification/Thawing media, immediately flush eyes/skin with water.  
If you have any problems caused by non-conformity to the present IFU.  
Thawing media contain the antibiotic gentamicin sulfate. Appropriate precautions should be taken to ensure that the patient is not sensitive to this antibiotic.  
NOTE: The long-term safety of vitrification technique and maximum storage in liquid nitrogen has not been established and unknown.

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