



# GM501 SpermMobil



1. Do not equilibrate GM501 SpermMobil in a CO<sub>2</sub>- incubator, just warm up to 37 °C.
2. To facilitate sperm activation add 1.50 - 2.00 µl GM501 Sperm Mobil to the sperm cells containing drop (approx. 30.00 - 40.00 µl/dilution 1:20) of processing media inside the Petri dish.
3. GM501 SpermMobil should be added to the opposing side from where the sperm cells are to be aspirated.
4. Wait for 10 minutes. The activating effect initiates after a few minutes and lasts approximately for one hour.
5. The dish should be placed on a heating plate at 37 °C during the diagnostic evaluation.

Gi140/V1

**Planer Limited**

110 Windmill Road, Sunbury-On-Thames  
Middlesex TW16 7HD, United Kingdom  
A Hamilton Thorne Company

**Tel:** +44 (0)1932 755 000  
enquiries@planer.com  
www.planer.com

**Manufactured by  
Gynemed GmbH & Co. KG**

Lübecker Straße 9 23738 Lensahn  
Germany

**Tel.:** +49 (0) 4363 90 32 90  
info@gynemed.de  
www.gynemed.de